




User Manual

OS-BE62036

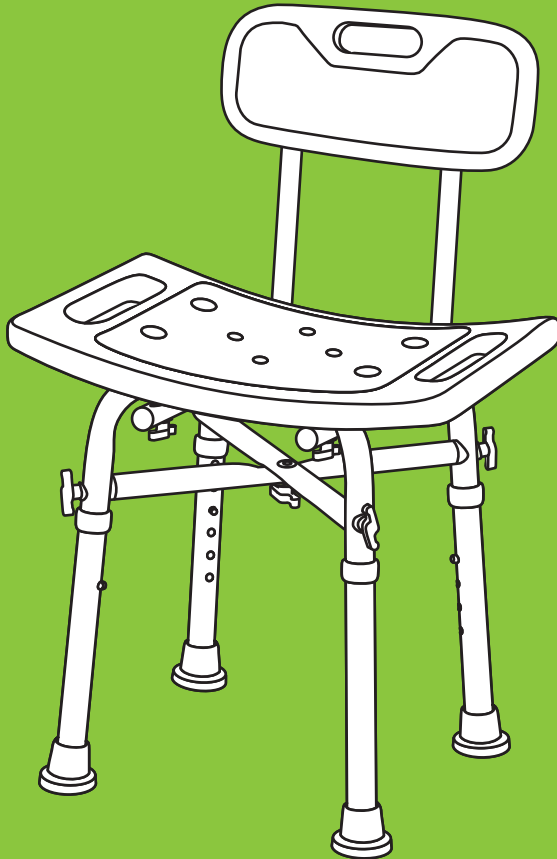


Please Read this Manual Carefully Before Using





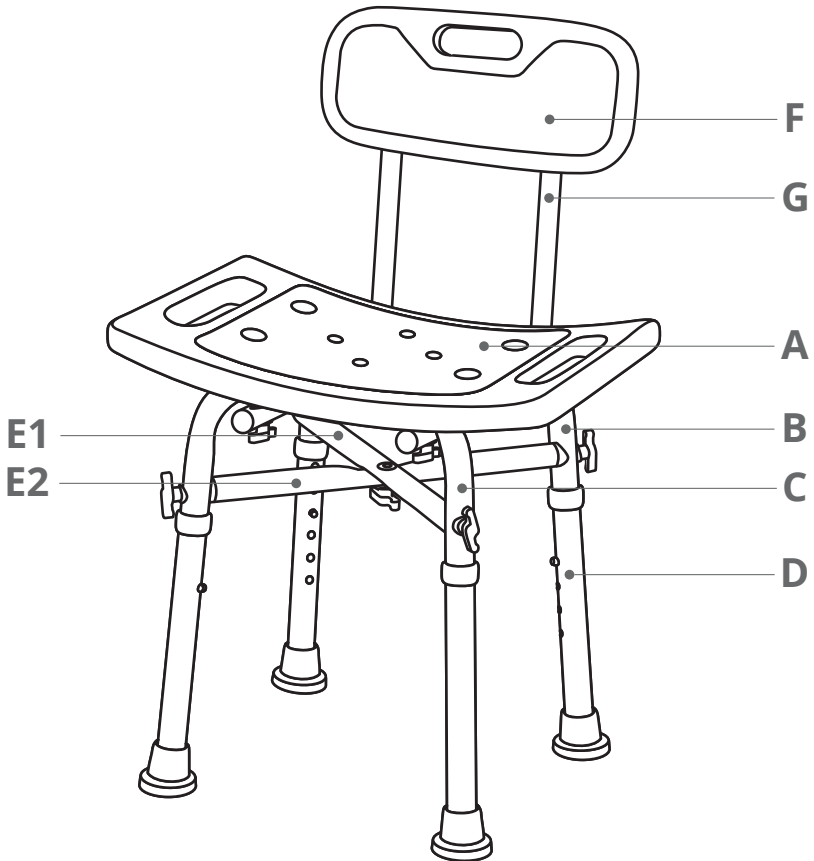


Thank you for choosing OasisSpace Shower Chair!

We have a professional Customer Service Support Team. Please feel free to contact us when you have any suggestions on our products or when you need any help. We will always be here for you and be responsible for our products.

01 Parts List




Number	Picture	Name	Quantity
A		Seat	1
B		Bottom Cross Leg Section	1
C		Top Cross Leg Section	1
D		Leg	4
E1		Cross Bar with Nut Inside	1
E2		Cross Bar	1

Number	Picture	Name	Quantity
F		Backrest	1
G		L-Shape Tube	2
H		25mm Screw & Washer	1
I		32mm Screw & Washer	5
J		45mm Screw & Washer	4
K		52mm Screw & Washer	4
L		Black Rubber Washer	4

02 Installation Instructions

Before Assembly

1. Please check the Parts List carefully and remove contents from carton. And then verify that all the parts are present. When missing parts or some parts are damaged, please contact us immediately.
2. We provide replacement parts to our customers if you need any replacement parts. Please provide the order number, part number, quantity, and pictures or video clips (if applicable) so that you can get what you need as soon as possible.

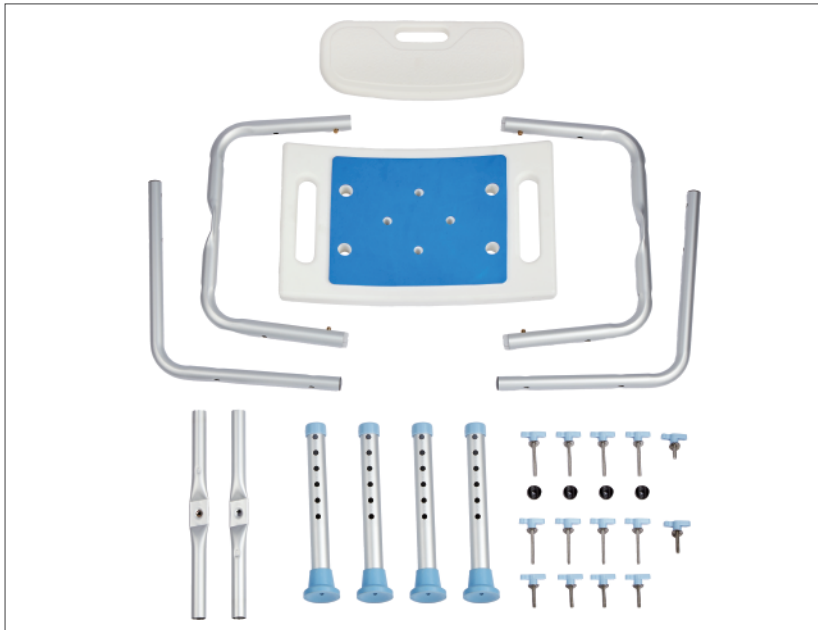


Assembly Instructions:


- Step 1.** Take out the Seat (Part# A), then you need to turn seat upside down and place the Bottom Cross Leg Section (Part# B) into the groove.




- Step 2.** Place the Top Cross Leg Section (Part# C) on top of section B, and fix them with the 32mm Screw & Washer (Part# I).

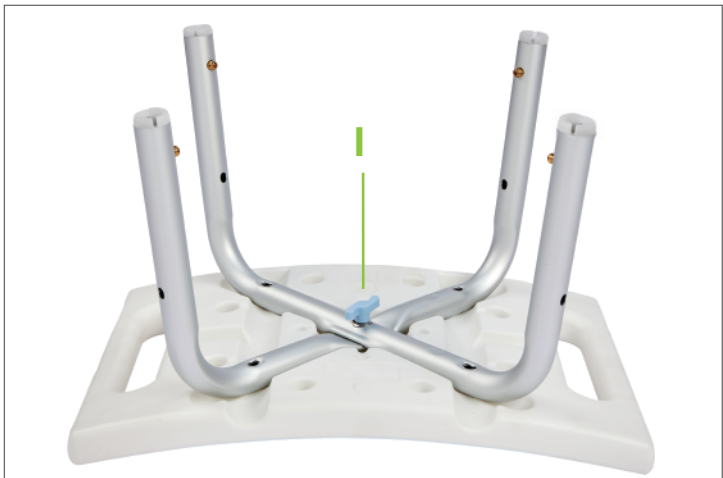



Step 3. Install the L-Shape Tube (Part #G) to the Leg Section with the 52mm Screw & Washer (Part #K) as shown in the picture.

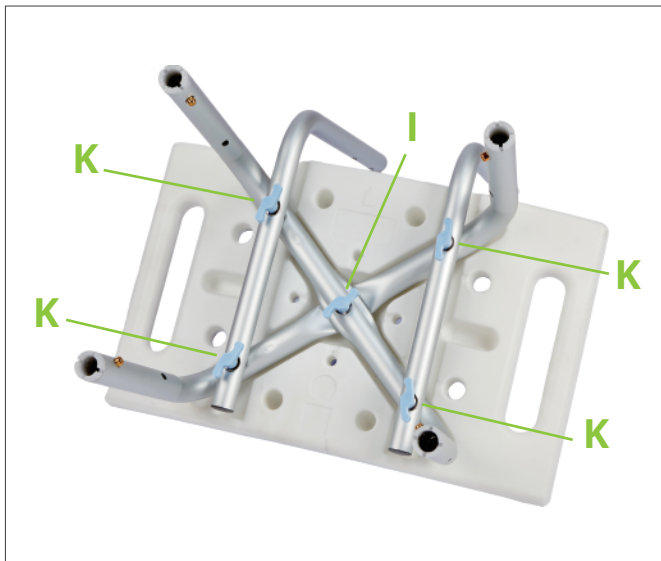

Step 4. First, put the Black Rubber Washer (Part #L) on the ends of the Crossbars (Part #E), then fix the crossbars with the 45mm Screw & Washer (Part #J).

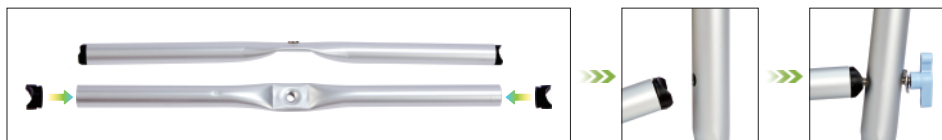

Step 5. Fix the two crossbars with the 25mm Screw & Washer (Part #H) as shown in the picture. Make sure that the positions of the two crossbars are as shown in the picture.

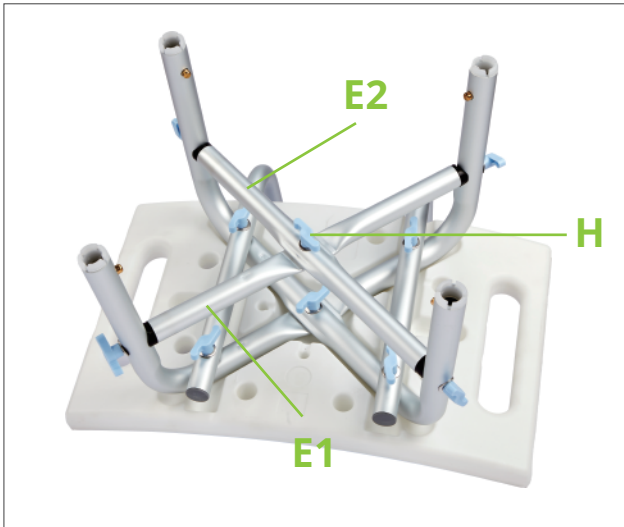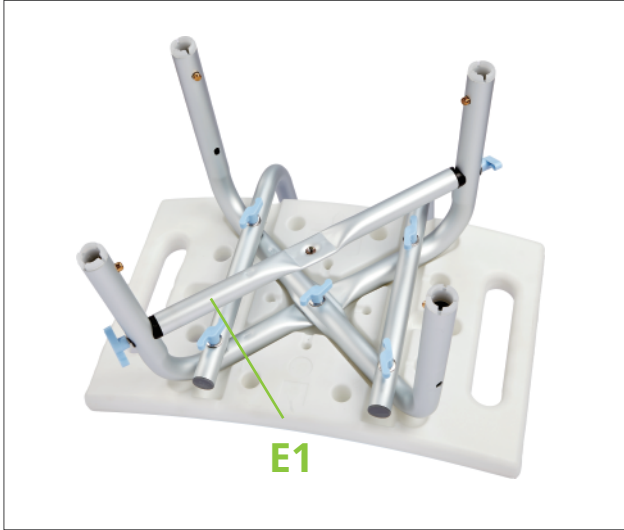

Step 6. Insert the Legs (Part #D) into the tube and adjust 4 legs to the same height.



Step 7. Finally, fix the Backrest (Part #F) to the L-Shape Tube with the 32mm Screw & Washer (Part #I).



Maintenance

1. Always keep the rubber discs clear of oil and dust.
2. Clean the rubber discs with standard household cleaning agents.
3. Never expose the rubber discs to sunlight for long periods otherwise they will quickly lose their elasticity.
4. Check the rubber discs for possible deterioration. If the suction cups show damage, replace immediately.
5. Please check whether screws and components are loose regularly. If they are loose, tighten them in time.

Additional Precautions

1. Inspect all parts for shipping damage before assembly. If there is shipping damage – **DO NOT USE**. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Ensure All thumb screws are tightened.
4. Check the rubber tips and wheels on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
5. All four leg extensions **MUST** touch the floor simultaneously at all times.

One Year Warranty

OasisSpace brand product is warranted to be free of defects in materials and workmanship for One Year for the original purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about OasisSpace device or this warranty, please contact us.



OasisSpace